

TECHNICAL DATA SHEET

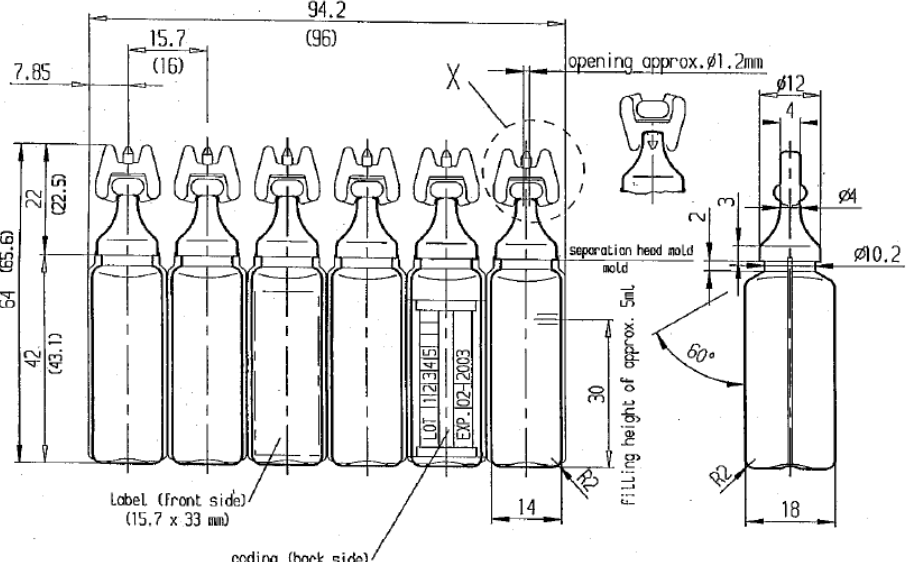
ISOMAR FLACONCINI IGIENE QUOTIDIANA

CLASS	Ila sterile (Rule 5) Invasive Medical Device
SIZE VARIANTS	24 bottles 5 ml each
PURPOSE AND EFFECT	<p>Isotonic seawater with respect to blood plasma, sterile contains a sterile solution of sea water diluted with purified water, until the concentration of 0.9% in sodium chloride. This medical device is used:</p> <ul style="list-style-type: none"> • To clear a stuffy nose in neonates, infants, children and adults, facilitating breathing; • To ensure the daily hygiene of the nasal cavity for colds, sinusitis, rhinitis, allergies to pollen and typically also for those who work indoors, especially in the presence of dust; • As a solvent of the mucus and blood residues of the nasal mucosa; • For the hygiene and cleanliness of the mouth and eyes of babies and infants; • For the hygiene cleaning of the nipples of nursing women; • In aerosol therapy to humidify the upper and lower respiratory tract; • For the washing of the ocular surface in the case of feeling of dryness and minor non-pathological disorders such as burning, redness and ocular fatigue induced by external factors such as exposure to wind and sunlight, dust, salt, dry environments, prolonged stay in front of screens video and use of contact lenses.
CHARACTERISTICS	<p>Isotonic Sea Water is contained in vials of polythene (PE) 5 ml. The product is sterile.</p>

CONFIDENTIAL INFORMATION FOR THE PROFESSIONAL BODY – DO NOT DISCLOSE TO THE PUBLIC

Coswell S.p.A.
Cap.Sociale 27.867.000 euro
C.F. 02827560729
P.IVA 00708541206
e-mail: info@coswell.biz
INTERNET: WWW coswell.biz

Sede legale Comm.le e Amministrativa
Via P. Gobetti, 4
40050 FUNO, Italy
Tel. 051/66.49.111 (13 linee ric.aut)
Fax 051/66.49.251
Registro imprese Bologna n. 02827560729

	
<p>ACTIVE PRINCIPLE</p>	<p>Isomar Isotonic Sea Water with respect to blood plasma contains a sterile solution of sea water diluted with purified water, until the concentration of 0.9% in sodium chloride.</p>
<p>HOW TO USE</p>	<ul style="list-style-type: none"> • NASAL APPLICATION: Insert the tip of the ampoule within the nasal cavity and lightly press. • EYE APPLICATION: Pour the solution on a sterile gauze pad and pass it gently on the eyelid. • FOR AEROSOL: Place the liquid into the aerosol according to the manufacturer's instructions. <p>The vial can be re-closed and re-used within 12 hours.</p>
<p>HOW TO STORE/ WARNINGS</p>	<ul style="list-style-type: none"> • Store in a cool dry place and avoid direct sunlight. • Use within expiry date.
<p>CONTRAINDICATIONS AND SIDE EFFECTS</p>	<p>Sterile non injectable solution. Keep out of the reach and sight of children.</p>

CONFIDENTIAL INFORMATION FOR THE PROFESSIONAL BODY – DO NOT DISCLOSE TO THE PUBLIC

Coswell S.p.A.
Cap.Sociale 27.867.000 euro
C.F. 02827560729
P.IVA 00708541206
e-mail: info@coswell.biz
INTERNET: WWW coswell.biz

Sede legale Comm.le e Amministrativa
Via P. Gobetti, 4
40050 FUNO, Italy
Tel. 051/66.49.111 (13 linee ric.aut)
Fax 051/66.49.251
Registro imprese Bologna n. 02827560729